LAICA®

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PS7126

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ELECTRONIC SMART BODY COMPOSITION SCALES FOR FULL BODY ANALYSIS, 30 POINTS OF DATA - INSTRUCTIONS AND WARRANTY

Dear customer, LAICA would like to thank you for choosing one of our products, designed according to strict performance and quality criteria in order to ensure total satisfaction.

IMPORTANT READ CAREFULLY BEFORE USE KEEP IN A SAFE PLACE FOR FUTURE REFERENCE

The instruction manual must be considered an integral part of the product and must be kept for its entire life. If the product is transferred to another owner, its documentation must also be transferred in its entirety.

To ensure the safe and correct use of the product, the user must carefully read the instructions and warnings contained in the manual insofar as they provide important information regarding safety, user and maintenance instructions. Should the instruction manual get mislaid or you require additional information or clarifications, please fill in the relevant form on the website: <https://www.laica.it/> in the FAQ and Support section.

Body composition scales measure weight and 30 points of data for body composition reporting based on height, age and gender. Compatible with bluetooth 4.0 and higher technology, it allows you to automatically send the data to your smartphone so you can monitor your progress at any time.

INDEX

| | |
|----------------------|---------|
| SYMBOLS KEY | page 10 |
| SAFETY WARNINGS | page 10 |
| BODY COMPOSITION | page 11 |
| PRODUCT DESCRIPTION | page 11 |
| INSTRUCTIONS FOR USE | page 12 |
| MAINTENANCE | page 13 |
| TROUBLESHOOTING | page 13 |
| DISPOSAL PROCEDURE | page 14 |
| WARRANTY | page 14 |

SYMBOLS KEY

Warning

Prohibition

SAFETY WARNINGS

- Before using, make sure the device is intact with no visible damage. If you have any doubts, do not use the device and contact the retailer.
- Keep the plastic bag used for packaging out of reach of children: suffocation hazard.

- This device must only be used for its intended purpose and in the manner indicated in the instructions.
- All other uses are considered improper and therefore hazardous.
- The manufacturer cannot be considered responsible for any damage due to improper or incorrect use.
- This device can be used by children on their own from the age of 8 upwards.
- Individuals with reduced physical, sensory or mental capacities or unskilled people must only use the product if previously instructed on its use, in safety and only if informed of the dangers connected to the product itself.
- Children must not play with the device.
- Cleaning and maintenance operations must not be performed by children without adult supervision.
- In the event of a failure and/or malfunction, do not tamper with the device.
- Always contact your retailer for repairs.
- Handle the scale with care, protect it against knocks, extreme temperature fluctuations, humidity, dust, direct sunlight and sources of heat.
- ⊘ Do NOT allow liquids to penetrate into the body of the scale.
This appliance is not waterproof. Do NOT leave and do NOT use the scale in water-saturated environments; humidity rates greater than 85% or contact with water or other liquids may cause corrosion or compromise proper scale operation.
- ⊘ Do NOT step onto the scale with wet feet, or if the surface of the scale is wet: slipping hazard.

ATTENTION! BEFORE USING THIS APPLIANCE

- The body fat, water, and skeletal muscle percentages and the basal metabolism calculated by the scale must only be considered to be indicative data. For further information, consult a doctor or dietician.
- Through the electrodes located on the platform and the hand bar, the scale sends a weak, low intensity electrical current that passes through the body.
The scale is not dangerous in any way.
- The scale is not recommended for the following individuals as the data detected by the scale may not be accurate:
 - individuals wearing a pacemaker or other electronic medical device
 - people with fever
 - children under 11 years of age and adults over 70 years of age
 - pregnant women
 - individuals undergoing dialysis, with oedema in the legs or osteoporosis
 - people who take cardiovascular medication

SAFE USE OF BATTERIES

- Remove the batteries if the scale is not used for long periods of time and keep them in a wellventilated and dry place, at room temperature.
- ⊗ Do NOT recharge the batteries if they are not rechargeable.
- ⊗ Do NOT recharge rechargeable batteries using methods other than those indicated in the manual or using unsuitable equipment.
- ⊗ NEVER expose the batteries to sources of heat or direct sunlight.
Failure to observe these instructions may result in damage to and/or the explosion of the batteries.
- ⊗ Do NOT throw the batteries into an open flame.
- The removal or replacement of batteries must always be performed by an adult.
- Keep the batteries far from the reach of children: swallowing batteries carries the risk of death. If batteries are swallowed, immediately contact a doctor.
- The acid contained in the batteries is corrosive. Avoid contact with the skin, eyes or garments.

BODY COMPOSITION

The body composition is made up of:

- lean mass: all components without fat (water, muscle, connective tissue, and other non-fatty organic components), bone, muscles, bowels, water, organs, and blood;
- fatty mass: all fats that can be extracted from adipose tissue and other tissues/organs;
- bone mass.

Through the 8 high precisions electrodes located on the platform and the hand bar, the scale sends a weak, low intensity electrical current that passes through lower and upper the body. The different masses respond differently to the current: lean mass provides less resistance than the opposite. This body resistance is called bio electrical impedance. The relative percentage of the various body masses varies based on gender, age, and physical stature.

Fitdays+™ app makes an automatic comparison of your measured data using gender, age, and physical stature versus reference data to create an analysis which includes: weight, BMI (body mass index), body fat %, fat mass, fat free body weight, muscle mass, muscle rate %, skeletal muscle %, bone mass, protein mass, protein %, water weight, body water %, subcutaneous fat %, visceral fat, BMR (basal metabolic rate), body age, ideal body weight, obesity level, body type, muscle in each limb, fat in each limb, abdominal muscle and abdominal fat (the 30 body composition data points).

For further information on these data points please visit:

www.laica.com/wp-content/uploads/PS7126_BC.pdf

This is not a medical device, the measurements cannot be used as medical diagnostic results. It must only be considered to be indicative data. For further information, consult a doctor or dietician.

PRODUCT DESCRIPTION

1. Measuring hand bar
2. Connecting wire
3. Display
4. Electrodes
5. Battery compartment

TECHNICAL SPECIFICATIONS

- Capacity: 180 kg / 400 lb
- Division: 100 g / 0.2 lb
- 4 disposable 1.5V AAA alkaline batteries
- LED display
- Operating frequency: 2.4G ISM Band (2402-2480 MHz)
- Environmental conditions for use: +10°C +40°C; RH 40%-80%
- Environmental storage conditions: +10°C +50°C; RH 40%-80%

The scale calculates the body composition for individuals aged between 11 and 70.

RECOMMENDATIONS FOR USE

- Always weigh yourself on the same scale at the same time, preferably when you wake up and while naked.
- To ensure the scale operates correctly, it is important to weigh yourself:
 - with dry, bare feet positioned correctly on the electrodes (see figure 2).
 - with fingers and thumb positioned correctly on the electrodes (see figure 2). Hold arms out at a 45 degree angle, avoiding contact with the stomach.
 - maintaining the correct posture until measurement is complete and display on scales stops flashing.
- Wait about 15 minutes after rising so water can distribute itself through your body.
- Scale precision could vary in the following cases:
 - a) high level of alcohol
 - b) high level of caffeine or drugs
 - c) after heavy meals
 - d) after rigorous sports activities
 - e) during illness
 - f) when pregnant
 - g) when dehydrated
 - h) with wet or unclean feet
 - i) swollen legs

INSTRUCTIONS FOR USE

This scale can be used in 2 ways:

- I. Body Weight Measurement only
- II. Body Weight and Body Composition Measurement with the **Fitdays+™** app.

I. Body Weight Measurement only:

(does not require pairing to smartphone application)

1. Place the scale on a flat, sturdy surface.
Note: An uneven or disjointed floor (e.g., rug, carpet, linoleum) may compromise the accuracy of the reading.
2. With the scale off, step onto it with your feet parallel and your weight evenly distributed. The scale will begin measuring. Do not move until the scale has measured your weight. The weight will flash 3 times.
Note: Do not lean on anything while the scale measures your weight to avoid altering the readings.
3. Step off the scale. The display will continue to show the reading for a few seconds before automatically turning off.

II. Body Weight and Body Composition Measurement with the **Fitdays+™** app:

(requires pairing to smartphone application)

When it is paired successfully, all body composition metrics will be available to be viewed on the **Fitdays+™** app.

FIRST TIME CONNECTING THE SCALE TO YOUR SMARTPHONE

The scale sends body composition data to the smartphone and make it available on the **Fitdays+™** app. **During the following operation, we recommend keeping the smartphone near the scale to keep the connection active.**

List of compatible devices:

- IOS 9.0 and above
- Android™ version 5.0 and above

1. Turn on bluetooth on your smartphone settings.
2. Download the free **Fitdays+™** app from App Store or Play Store™.
You can also scan the QR code with your smartphone's camera to direct you to the app in the Apps store.
3. Launch the **Fitdays+™** app, then step onto the scale.
4. Follow the instructions in the app to pair the LAICA scale and to create a new user.
If the scale is paired successfully with the app, you will see the weight measuring on the app.

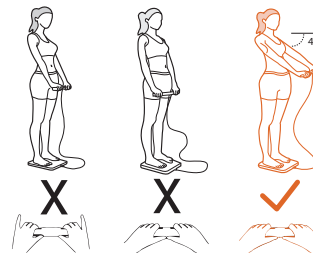
If the weight is not measuring on the app, please make sure the device is paired by trying the following:
In the app, go to **Device**. You should see an entry of "**LAICA PS7126**". If not, tap on **Add Device** to add a device.
Step on the scale and tap on **Scale → Searching bluetooth for pairing**. If the scale is found, it will appear with an option to connect. Tap on "**LAICA PS7126**" device, follow the on screen instructions to bind the device in the app.

In order to ensure the accuracy of measurement data, please fill in your basic information correctly (height, birthday, gender).
The icon of **Fitdays+™** will automatically change from  to  after the device once connected to your smart phone.
Each time the scale is moved and then returned to its resting surface, it will automatically re-calibrate. Step on the scale to turn it on, wait for the weight to return to 0.0 kg/lbs/st.

HOW TO MEASURE YOUR BODY COMPOSITION

1. Launch the **Fitdays+™** app, keep the smartphone near to the scale to maintain connection. Go to **Device**, select "**LAICA7126**", then "**Go to Measurement**", the app is ready to receive data from the scales.
2. Place the measuring hand bar on the scale, pressing the center of scale to turn it on and wait for weight to return to 0.0kg/lbs/st. This ensures that the measuring hand bar is not included in the weight measurement.
3. Pick up the measuring hand bar, taking care to position your bare feet on the electrodes, and hold the measuring hand bar correctly (check the picture on the right).
4. After few seconds, the display on the scales will stop flashing and your body composition data will be available in the **Fitdays+™** app of your smartphone.

More than one account may be set up in the **Fitdays+™** app. The scale has an automatic recognition function. When the weight is within +/- 2 kg difference the app will remind you to choose the correct user.

MAINTENANCE

BATTERY REPLACEMENT

When the "LO" or battery symbol appears on the display, replace the batteries with the same type.

1. Open the battery compartment by pulling the tab on the cover.
2. Remove the worn batteries and dispose of them according to the instructions in the paragraph "Disposal Procedure".
3. Insert the new batteries according to the polarity markings.
4. Close the battery compartment.

MAINTENANCE

- Store the device in a cool, dry area.
- Store the device in a horizontal position.
- Only clean the scale with a soft, dry cloth.
- Never use chemicals or abrasive products.

TROUBLESHOOTING

| Problem | Possible cause | Solution |
|---|---|---|
| The scale doesn't turn on. | You have not stepped onto the scales properly. | Step onto the scale with your feet parallel. Note: the scale will turn on only after stepping onto it. |
| | The batteries have run out. | Replace the batteries. |
| "Lo" or the battery symbol is shown on the display. | The batteries are low. | Replace them. |
| The display shows "Err". | Weight greater than the scale capacity. | Step off the scale immediately so as not to damage it. The maximum capacity is 180kg/400lbs/28st |
| The display shows "C". | Error occurred during measurement. | Step off scales for 5 seconds, step back on the repeat measuring process. |
| The "Fitdays+™" app doesn't display any data. | The connection has not been enabled in the smartphone settings. | Connect your smartphone. |
| | The smartphone went into standby during weighing. | Keep the app active on your smartphone during weighing to check that data is sent instantly. |
| The "Fitdays+™" app only shows your body weight. | You did not step on the scale with bare feet or did not contact the electrodes. | Step onto the scale with bare feet and make sure your feet and hand contact the electrodes correctly. |

All weights are approximate having been rounded up or down to the nearest whole number.

We guarantee accuracy within +/-1% + 1 division.

Further user and troubleshooting support is available in the FAQ section of the **Fitdays+™** app.

DISPOSAL PROCEDURE

The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment (**Dir. 2012/19/Eu-WEEE**).

At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m² without having to purchase a new, similar device.

This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same. Caution! The wrong disposal of electric and electronic equipment may involve sanctions.

To correctly dispose of batteries (**Dir. 2013/56/Eu, the Waste Batteries and Accumulators Regulations 2009**) do not throw them into household waste but rather dispose of them as special waste at specialised recycling centres. For more information about disposal of used batteries, contact the store where you purchased the equipment containing the batteries, the municipality or the local waste disposal service.

This product is meant for household use. The conformity, certified by the CE-marking on the device, refers to the Directive 2014/30/Eu on electromagnetic compatibility and on Directive 2011/65/Eu on the restriction of the use of certain hazardous substances in electrical and electronic equipment.

This product is an electronic device that was tested to assure, according to the current technical knowledge, that it does not interfere with other nearby located equipment (electromagnetic compatibility) and that it is safe if used according to the advices given in the instructions for use. In case of operation anomalies, do not keep using the device and, if necessary, refer to the manufacturer directly.

The manufacturer, LAICA S.p.A., declares that the device is compliant with Directive 2014/53/Eu. The complete statement of compliance is available at the following Internet site: www.laica.it.

This product is meant for household use. The conformity, certified by the UKCA-marking on the device, refers to the Electromagnetic Compatibility Regulations 2016 and The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012. This product is an electronic device that was tested to assure, according to the current technical knowledge, that it does not interfere with other nearby located equipment (electromagnetic compatibility) and that it is safe if used according to the advices given in the instructions for use. In case of operation anomalies, do not keep using the device and, if necessary, refer to the manufacturer directly.

The manufacturer, LAICA S.p.A., declares that the device is compliant with Radio Equipment Regulations 2017. The complete statement of compliance is available at the following Internet site: www.laica.it.

WARRANTY

This appliance is guaranteed for 2 years from the time of delivery of the goods, or another longer term envisaged by the national legislation of the consumer's residence. This provision complies with Italian and European legislation. The LAICA products are designed for home use and must not be used in public venues. The warranty only covers manufacturing defects and does not apply if the damage is caused by an accidental event, incorrect use, negligence or misuse of the product. Use only the accessories supplied; the use of different accessories may result in invalidity of the warranty. Do not open the unit for any reason; in the case of opening or tampering, the warranty is definitively voided. This warranty does not apply to parts subject to wear or to the batteries when supplied. After 2 years from delivery, or another longer term envisaged by the national legislation of the consumer's residence, the warranty expires; in this case, the technical assistance interventions will be carried out against a fee. Information on technical assistance, whether under warranty or for a fee, can be requested by contacting info@laica.com.

No payment will be due for repairs or replacements of products that fall within the terms of the warranty. In the event of faults, contact the retailer. Do NOT send the appliance directly to LAICA. All the operations under warranty (including those of replacement of the product or part thereof) will not prolong the duration of the original period of warranty of the product replaced. The manufacturer declines any liability for any damage that may, directly or indirectly, be caused to persons, property or animals as a result of the non-observance of all the requirements established in the relevant instructions manual and concerning, especially, warnings relating to installation, use and maintenance of the appliance. LAICA, in its constant commitment to improving its products, is entitled to changing without any notice, in whole or in part, its products in relation to production requirements, without this entailing any liability for LAICA towards its dealers. For further information: www.laica.it.

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LC182 - 01/2025

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by LAICA S.p.A. is under licence.

Other trademarks and trade name are those of their respective owners.

iOS, iPhone, iPad mini, iPod touch, App Store are trademarks of Apple Inc., registered in the U.S. and other countries - Android™, Play Store™ and Google Play™ are trademarks of Google Inc.

LAICA®

LAICA S.p.A.

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36048 Barbarano Mossano (VI) - Italy

Tel. +39 0444.795314

info@laica.com

Made in China

UK Importer: Strix Ltd, Forrest House, Ronaldsway, Isle Of Man, IM9 2RG, British Isles

www.laica.it